



DEVICE DETAILS

| | |
|--------------------------------------|--|
| NAME OF DEVICE | WRIST DIGITAL BLOOD PRESSURE MONITOR |
| ESTABLISHMENT NAME | REVO HEALTHCARE SDN BHD |
| REGISTRATION NO | GB7946820-40403 |
| DESCRIPTION | The A&D Wrist Digital Blood Pressure Monitors use an oscillometric method, and detects pulse rate and determines blood pressure by pressurizing a cuff around the wrist. When the start button is pressed, the device detects pulse rate during inflation, and decides pressurization setting value, and automatically inflates up to the pressurization value. If the pre-set pressurization is not enough to detect patient's BP value, the device automatically repeats re-inflation continuously. The device also has constant deflation, automatic quick deflation and other safety functions to make it easier to operate. |
| DEVICE GROUPING TYPE | Family |
| INTENDED PURPOSE | The A&D Wrist Digital Blood Pressure Monitor is intended to measure the systolic, diastolic and pulse rate of an individual by using noninvasive technique in which an inflatable cuff is wrapped around the wrist. |
| VALIDITY DATE OF REGISTRATION | 25-02-2020 - 24-02-2025 |

LIST OF DEVICE



| # | NAME OF DEVICE | IDENTIFIER |
|---|---|------------|
| 1 | UB-525 Wrist Digital Blood Pressure Monitor | UB-525 |
| 2 | UB-533 Wrist Digital Blood Pressure Monitor | UB-533 |
| 3 | UB-533PGMR Wrist Digital Blood Pressure Monitor | UB-533PGMR |